

Blood Glucose Test Strip

GLUCOCARD S Test Strips

with aluminum foil packet individually



Foil packet packaged individually
GLUCOCARD S Test Strip provides you
with more convenience and safety with
GLUCOCARD S / S onyx.



Aluminum foil packet
Individually packaged

Advantages

- ✓ Not easily affected by humidity
- ✓ Minimizes the effects of common interferences
- ✓ Convenient to distribute and carry
- ✓ Easy to manage quantity
- ✓ Safely disposable without touching blood.

GLUCOCARD S Test Strips

with aluminum foil packet individually

Operation



Use one foil packet



Tear the foil packet from the notch



Take the test strip out of the foil packet



GLUCOCARD S

or

GLUCOCARD S onyx



Insert the test strip into the meter's slot



Apply a drop of blood and start the measurement



Blood glucose levels are then displayed




Dispose of the test strip using the opened foil packet

Applicable meter models

GLUCOCARD S and GLUCOCARD S onyx

Data Management Application

BG Logger (data management application) will be available as a free download at the App Store or Google Play.

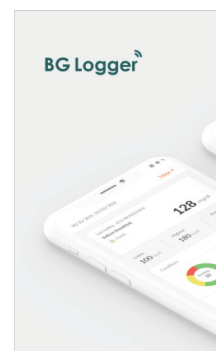

Design and specifications may be changed without prior notice.

ARKRAY GLOBAL BUSINESS, INC.

Yousuien-nai, 59 Gansuin-cho,
Kamigyo-ku, Kyoto 602-0008, JAPAN

TEL 81-75-662-8979 FAX +81-75-431-1202